

BeHear® PROXY User Manual



Sound Enhancement by Alango Technologies, Ltd.

Wear & Hear is the brand name for a line of concept, future-looking wearable audio devices that are developed by Alango Technologies, a leading supplier of voice and audio enhancement technologies for the communication and entertainment markets.

BeHear and Wear & Hear are registered trademarks of Alango Technologies, Ltd.

For more information, contact: Wear & Hear c/o Alango Technologies, Ltd. 2 Etgar St. Tirat Carmel, ISRAEL 3903213

Tel.: +972 (0)77 320-4102 info@WearAndHear.com support@WearAndHear.com

www.WearAndHear.com

©2018-2021 Alango Technologies, Ltd.

Contents

Introduction	5
Device Overview	7
Basic Operation	9
Charging the Battery	9
Turning the Neck Speaker ON and OFF	9
Checking the Battery Status	9
Checking the Battery Level using the Neck Speaker	9
Checking the Battery Level using the Application	9
Wearing Instructions	10
Fitting the Ear Buds to Your Ears	10
Getting Started	11
Pairing with Bluetooth Devices	11
How to Place the BeHear PROXY Neck Speaker into Pairing Mode	11
Pairing BeHear PROXY with Additional Bluetooth Devices	11
Connecting with Bluetooth Devices	11
Automatic Connection of Paired Bluetooth Devices	11
Choosing from Multiple BeHear PROXY Devices Paired to the Same Smartphone	11
Connecting Simultaneously to Two Different Bluetooth Devices	11
Selecting Operation Mode	12
Checking for Updates	13
Personalization	13
Hearing Assessment	13
Before Starting the Hearing Assessment	14
Conducting the Hearing Assessment	14
Hearing Assessment Results	15
Using BeHear PROXY	16
Listening to Audio	16
SonicVibrance – Stereo Sound Enhancement Function	16
ListenThrough™ – Stay Safe & Alert Function	16
Handling Mobile Phone Calls	18
EasyListen™ – Slowing Down Incoming Speech	18
Personal Hearing Amplification	19
Managing Personal Hearing Using the Neck Speaker Controls	19
Managing Personal Hearing Using the W&H BeHear App	19
Advanced Sound Tuning	20
Mobile Application	21
Installing the Application	21

First Launch	21
Connection	22
Connection with an Android Phone	23
Connection with an iOS Phone	24
Home Screens	26
ListenThrough Audio Player State	26
SonicVibrance Audio Player State	26
Phone Call State	26
Personal Hearing Amplification State	27
Idle State	27
Advanced Sound Tuning for Personal Hearing Amplification State	28
Personalization	29
Personal Hearing Assessment	29
Edit Audiogram	31
Library of Audiograms	31
Settings	32
Application Settings	33
Device Settings	34
Update	37
Information	38
Introduction	38
Tutorials	39
Version	39
Terms of Use	39
FAQ	40
Support	40
Troubleshooting	41
General Issues	41
Reset the Neck Speaker	41
Updating the Neck Speaker	41
Legal Information	42
FCC Compliance Statement	42
EU Declaration of Conformity	43
Furonean Union – Disposal Information	43

Introduction

BeHear PROXY is a hands-free neck speaker with built-in, personalizable sound enhancement for TV watching, music listening, mobile calls, and world sounds. It can be used with hearing aids.

BeHear PROXY combines the functionality of three devices into one:

- Bluetooth stereo neck speaker for wireless mobile phone calls and media playback
- Assistive hearing device enhancing intelligibility of speech and other important sounds
- Digital hearing aid for personalized sound amplification across a wide sound spectrum

BeHear PROXY utilizes voice processing technologies developed by Alango Technologies, Ltd., such as:

- ListenThrough™ passing through important ambient sounds during music playback
- **SonicVibrance™** extending stereo sound, enriching music impression
- EasyListen™ slowing down speech dynamically during phone conversations
- Personal Sound Amplifier providing dynamic, compressed amplification of ambient sounds
- Hearing enhancement functions including noise reduction, echo cancellation, and more

While the neck speaker itself includes a variety of controls to manage its functions, the W&H BeHear mobile application is also provided to complement these controls. The application allows the wearer to customize the speaker for his/her specific hearing capabilities and preferences, and further personalize hearing based on real-time conditions. The application supports:

- A personal hearing assessment for automatic adjustment of the sound in each of the operational modes – ambient hearing, phone calls, and audio play – to suit the capabilities and preferences of the wearer
- Fine tuning the sound in real time
- Management of assistive listening functions during phone calls and media playback
- Various device settings, including default mode behavior, preferences for control, voice, tone and vibration alerts, and more

The W&H BeHear application can be downloaded from the Google Play Store or Apple Store:









Product No.: ABH-130

The BeHear PROXY neck speaker works in two different operational modes:

Loudspeaker (cables are retracted into neck speaker):

- Bluetooth audio playback is supported, with or without SonicVibrance activated (but without ListenThrough).
- Mobile calls are supported, with or without EasyListen activated.




Earphones (cables are extracted and buds are placed in ears):

- Personalized sound amplification is supported.
- Bluetooth audio playback is supported, with or without ListenThrough activated (but without SonicVibrance).
- Mobile calls are supported, with or without EasyListen activated.



Device Overview

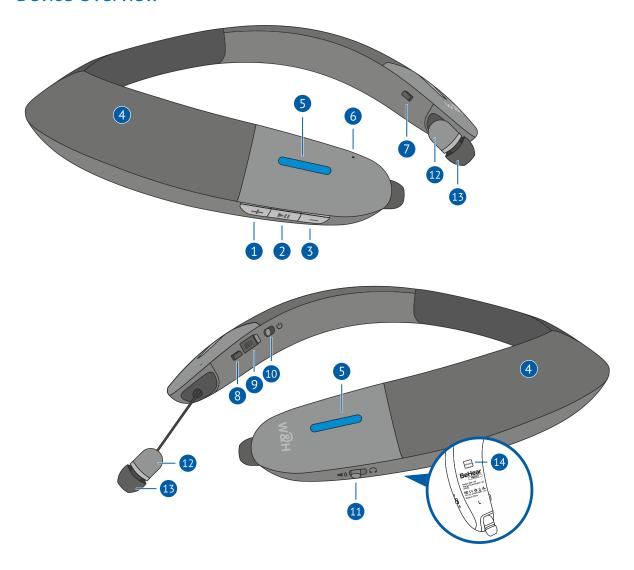

	Icon	Component	Function	
1	+	Volume up button	Short press to inc	rease call/audio/ambient volume. (*)
2	▶ II	Multi-functional button	Handle phone calls (read <u>Handling Phone Calls</u>) Handle listening to audio (read <u>Listening to Audio</u>) Handle personal hearing (read <u>Personal Hearing</u>) (**)	
3		Volume down button	Short press to decrease call/audio/ambient volume. (*)	
4		Loudspeakers (left and right)	Loudspeakers are located under the fabric covers.	
5		Notification LED (left and right)	Flashing red and blue	Neck speaker is in Bluetooth pairing mode
			Flashing blue	Neck speaker is connected to the Bluetooth device
			Flashing red	Neck Speaker is not connected to the Bluetooth device, or the battery is low
			Flashing purple	Personal hearing is active
			Steady red	Neck speaker is being charged
			Steady green	Neck speaker is charged (battery level >50%)
			Light blue	Neck speaker is powered on/off (toggle)

	Icon	Component	Function
6		Microphone	Embedded microphone for phone call and ambient sound amplification.
7		Left retract button	Press and hold to retract left earphone
8		Right retract button	Press and hold to retract right earphone
9	•<-	Micro-USB port	Open the cover to insert the supplied cable to charge the battery.
10	G	Power button	Shift and hold to turn BeHear PROXY on and off. Shift briefly to show battery level (when BeHear PROXY is on).
11	P (Mode switch	Switch between loudspeaker and earphone operation modes.
12		Earphones (left and right)	Pull the earphones gently to extract the cables and then place them in your ears.
13		Silicone ear buds	Mushroom type; extra-small, small, medium, large sizes Conical type; small, medium, large sizes Tulip (open fit) type; medium size Tube (open ear) type; extra-small size
14		Serial number label	Unique number identifying the BeHear PROXY neck speaker in the Connection screen of the W&H BeHear smartphone application

Note: You can modify functions for each button to make it convenient for frequently-used actions and avoid unintentional changes.

- (*) Additional functions can be enabled for this button using the W&H BeHear smartphone app as described in the <u>Using BeHear PROXY</u> section.
- (**) This button's function can be disabled using the W&H BeHear smartphone app.

Basic Operation

Charging the Battery

It takes approximately three hours to fully charge the battery. We recommend charging the battery before using your BeHear PROXY neck speaker for the first time.

- Plug the larger end of the supplied cable into a standard AC outlet USB charger adapter, or into the USB port of a computer.
- 2. Plug the cable's smaller end into the Micro USB port located in the BeHear PROXY.
- 3. A vibration in the neck speaker indicates that charging has started. During charging the LED on the neck speaker will be red, turning green when BeHear PROXY is fully charged.



Note: The BeHear PROXY neck speaker should not be used while charging.

Turning the Neck Speaker ON and OFF

To turn on BeHear PROXY:

Shift and hold the Power switch until the unit vibrates and the LED turns light blue. Release the switch. It will return to its original position automatically.

To turn off BeHear PROXY:

Shift and hold the Power switch until the unit vibrates and the LED turns light blue. Release the switch. It will return to its original position automatically.



Note: Vibration can be disabled by changing a setting in the W&H BeHear application.

Checking the Battery Status

You can view an estimate of the remaining battery charge in either of two ways:

Checking the Battery Level using the Neck Speaker

When the device is powered on, quickly shift and release the Power switch. The notification LED flashes for 1-2 seconds indicating battery level, as follows:

- If the LED turns green the status is good (more than 50% battery charge remains).
- If the LED turns **yellow** the status is fair (battery charge level is between 10% and 50%).
- If the LED turns **red** the status is low (below 10%) and the neck speaker needs to be recharged.

Checking the Battery Level using the Application

When the device is powered on, view the level in the battery icon which appears at the top of each screen. The more white



filler you see, the more charge is left in the battery. The amount of charge left is also displayed as a numerical value measured in percentage.

Note: During neck speaker use, if the battery level drops below 10% the LED will flash red and a voice alert will be heard periodically.

Wearing Instructions

BeHear PROXY is a neck loop style device where the flexible silicon neckband should be placed behind your neck and the hard plastic collar should rest on your shoulders.

The earphones at the ends of the cables can be extracted from the collar and inserted into your ears.

Wear your BeHear PROXY neck speaker as described below:

1. Place the BeHear PROXY neckband behind your neck such that the hard plastic collar ends are on your chest and the fabric covers of the speakers face up.



Pull the earphones gently to extract the cables. Insert the
earphones into your ears. Your BeHear PROXY package contains
several sizes and types of silicone ear buds, and you may have to
experiment to find the buds that fit your ears most comfortably for
continuous wear.





Fitting the Ear Buds to Your Ears

Your BeHear PROXY neck speaker comes with four shapes of silicone ear buds: conical, mushroom, tulip (open fit), and tube (open ear) in up to four sizes for each shape. The most common type and size of buds has already been attached to the earphones. However, if the buds do not fit snugly and comfortably in your ears you should remove the silicone covers from the earpieces and try different options, as follows:

1. Peel off the silicone ear bud from the earpiece.

Place the replacement ear bud over the earphone speaker.



Getting Started

Pairing with Bluetooth Devices

Before you can start using your BeHear PROXY neck speaker with a Bluetooth device (such as a smartphone), you need to pair it with the Bluetooth device. Pairing can be done in either of two ways:

- Using the W&H BeHear app (which can be downloaded for free from the Google Play Store
 or the Apple App Store, see "Installing the Application" later in this manual). After launching
 the app and placing the neck speaker into pairing mode (see instructions below), the pairing
 process will proceed. Follow to the application's prompts (see "Connection" below).
- Using your smartphone's Bluetooth setup process. (This is mandatory for iOS devices see
 the "Connection with an iOS Phone" section for detailed instructions.) Make sure the
 Bluetooth function is turned on in the Bluetooth device. The neck speaker will appear in the
 list of available devices as BeHear PROXY.

How to Place the BeHear PROXY Neck Speaker into Pairing Mode

- 1. Make sure the BeHear PROXY neck speaker is powered OFF.
- 2. Then, shift and hold the power button \circlearrowleft on the collar for **up to 5 seconds**, until the LED alternates red/blue.
- 3. When the pairing is complete the BeHear PROXY neck speaker will be connected to the Bluetooth device automatically, as indicated by the blue LED which blinks periodically every 5 seconds.

Pairing BeHear PROXY with Additional Bluetooth Devices

After pairing BeHear PROXY with the first Bluetooth device repeat the same steps to pair it with additional Bluetooth devices. Your BeHear PROXY neck speaker can be paired with up to eight Bluetooth devices in total. If the list of paired devices is full you can clean it up by resetting the neck speaker as described in the Reset the Neck Speaker section of this manual.

Connecting with Bluetooth Devices

Automatic Connection of Paired Bluetooth Devices

BeHear PROXY remembers and can connect automatically with the last two connected devices.

Note: When you pair and connect BeHear PROXY with a third device, the first paired device becomes disconnected.

Choosing from Multiple BeHear PROXY Devices Paired to the Same Smartphone

If you have more than one BeHear PROXY neck speaker paired with the same phone you can choose the desired neck speaker by its serial number. The serial number of the BeHear PROXY neck speaker is listed underneath the product name in the W&H BeHear app **Connection** screen. It is also printed on the underside of the BeHear PROXY neck speaker.

Connecting Simultaneously to Two Different Bluetooth Devices

Your BeHear PROXY neck speaker supports multipoint connection mode, managing two Bluetooth connections simultaneously. For example, if you connect your neck speaker with two phones or with a phone and the **HearLinkTM PLUS TV transmitter** (a Wear & Hear companion product, sold separately), you can receive calls or listen to audio from either device without having to disconnect and reconnect.

RECOMMENDATION:


If a long time has passed between uses, shift the BeHear PROXY neck speaker Power button briefly to initiate reconnection to the paired Bluetooth device – HearLink PLUS transmitter or/and the smartphone.

BeHear PROXY neck speaker supports two typical connection use cases:

1. Your BeHear PROXY neck speaker is connected simultaneously to a smartphone and to the HearLink PLUS TV transmitter. The TV audio can be transmitted to your BeHear PROXY neck speaker, which will also be available to receive/initiate a phone call.



2. Two BeHear PROXY neck speakers are connected simultaneously to the same HearLink PLUS TV transmitter. Both users can be listening to TV audio at the same time.

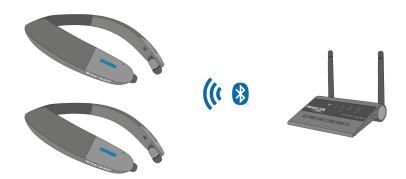

RECOMMENDATION:

Simultaneous operation of two BeHear PROXY neck speakers may be affected by nearby Bluetooth equipment.

Choose **APTX** or **SBC** audio CODEC in **Settings->Device** menu of the **W&H BeHear** app if you experience sound distortion in one or both neck speakers.

Selecting Operation Mode

Your BeHear PROXY neck speaker operates in one of two modes:

- Loudspeaker mode for open ear listening and mobile phone calls
- **Earphone mode** for private listening, mobile phone calls, and personal amplification of ambient sounds

Use the mode switch to set the desired operation mode, as follows:


- To select **Loudspeaker** mode set the switch to the **◄)** position.
- To select **Earphone** mode set the switch to the position. Extract the cables and insert the earphones into your ears.

Checking for Updates

Keeping BeHear PROXY updated with the latest version of the firmware allows you to enjoy its most advanced features. We recommend checking for updates before you start using your neck speaker.

To check for an update:

- 1. Make sure the BeHear PROXY neck speaker is powered on and connected to a phone with the **W&H BeHear** application installed.
- 2. From the bottom menu of the app, choose the **Settings** option and tap the **Update** button in the screen.
 - **Note**: The app checks for updates automatically when the **Auto** option is selected for the **Headset Update Check** in the **Application** section of the **Settings** screen.
- 3. If an update is available, launch the update process by following the instructions displayed on the screen.

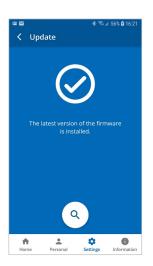

RECOMMENDATION:

The firmware update process lasts for several minutes (typically 15-20) during which time the neck speaker and application must stay powered on and connected. We recommend verifying that the batteries in both the neck speaker and the phone be sufficiently charged prior to starting the update.

Personalization

We highly recommend that you adjust the acoustic performance of the neck speaker to suit your own personal hearing capabilities and preferences, using the built-in **Hearing Assessment** in the **W&H BeHear** app.

If you have a recent audiogram you can skip the W&H BeHear Hearing Assessment and enter the values from it manually. Tap the **Personal** option in the bottom menu and enter the values into the table in the **Editor**.

You can also choose any of the audiograms available in the **Library** of the application.

The results (whether generated from the hearing assessment, chosen from the library, or manually input into the editor) are automatically applied to BeHear PROXY neck speaker to create profiles that reflect your own hearing abilities. These personalized profiles ensure that you get the most benefit from BeHear PROXY, using it as a truly personal sound amplifier. Sounds from music play and phone calls are personalized as well.

Hearing Assessment

During the assessment, six different testing sounds will be played for each ear, one after the other. Each testing sound represents a different range of audible frequencies. The purpose of the assessment is to determine the best volume for each frequency range based on your unique hearing profile. The results will be saved in the app's **Library** and applied to the neck speaker. They will be used to adjust the audio performance of your personal BeHear PROXY neck speaker.

Two variances of the assessment are available:

- **Basic** assessment. This is the default option, which will be sufficient for most users. During the Basic assessment you will be asked to indicate at which volume the testing sound first becomes audible (**Hearing Threshold**).
- Advanced assessment. This assessment is more comprehensive and therefore takes longer.
 During the Advanced assessment you will need to indicate two different volume levels for each testing sound. One is the Hearing Threshold, the same as for the Basic assessment.

The other is the **Comfortable Level**, which reflects the volume level at which it is most comfortable for your ear to process the testing sound.


In order to toggle between the Advanced and the Basic assessment tap the **Settings** icon at the bottom of the **W&H BeHear** app screen and select the **Application** option. Scroll down to the **Hearing Assessment Mode** option.

Before Starting the Hearing Assessment

Before starting the assessment, you should verify the following:

- You have installed and launched the W&H BeHear app on your smartphone.
- You have selected the desired assessment mode Basic (default) or Advanced, in the Application section of the Settings screen.
- The BeHear PROXY neck speaker is turned on and connected to the smartphone via Bluetooth.
- The BeHear PROXY neck speaker is set to the earphone mode.

Note: The hearing assessment can be conducted only when the neck speaker is in the earphone mode (the mode switch is in position).

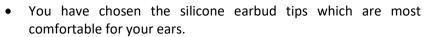

- You are wearing your BeHear PROXY neck speaker, and the earphones have been placed firmly in your ears.
- You are in a quiet environment.

Note: The **W&H BeHear** app monitors the ambient noise during the assessment and will not allow the assessment to proceed if the ambient noise level is too high.

Conducting the Hearing Assessment


To start the assessment, tap the **Personal** icon at the bottom of the **W&H BeHear** app screen and select the **Assessment** option. Tap the **START** button at the bottom of the screen.

The hearing assessment starts with the left ear, goes through six different frequencies, and then repeats with the right ear. Advancing to the next frequency (by pressing the SAVE button in the upper right-hand corner of the app) can be done only after completing the previous one.

If you use the **Basic** assessment:

- Set the Hearing Threshold level by placing your finger on the round button on the scale and sliding it up until you clearly hear the sound. Then, slowly move the button down to the lowest level at which you can still hear the sound. You can do this as many times as needed to find the correct sound threshold level for your ear.
- 2. Once the slider is set to the desired position, tap the **SAVE** button at the top right corner of the screen.
- 3. Repeat that action for the five remaining testing sounds for the left ear, and then repeat the process for the right ear.
- 4. Save the results, as instructed by the app prompts.

Note: The "\\" and "\\" buttons on the screen as well as the volume buttons on the phone can also be used to adjust the volume of the testing sound.

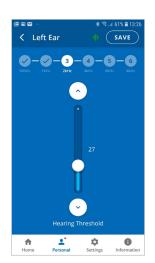





If you use the **Advanced** assessment:

- 1. Set the **Hearing Threshold** level by placing your finger on the round button on the scale and sliding it up until you clearly hear the sound, then slowly move it down to the lowest level at which you can still hear the sound. You can do this as many times as needed to find the correct sound threshold level for you.
- 2. Move the **Comfortable Level** slider to find the most comfortable listening level for the testing sound in the left ear (neither too loud nor too soft).

Note: The **Comfortable Level** will always be higher (louder) than the **Hearing Threshold**.

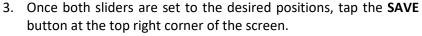

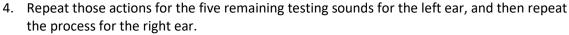



Note: Before saving the assessment results you can return at any time to the previous stages of the assessment to verify and modify your input. Simply tap the relevant frequency indicator at the top of the screen to review and/or repeat the assessment for that frequency.


Hearing Assessment Results

Results of your hearing assessment are stored in the **W&H BeHear** app's **Library** under a unique name. You can see them by tapping the **Personal** icon at the bottom of the app, selecting the **Library** option, and choosing the relevant assessment.

The assessment can be re-taken at any time.

Note: Only the most recently applied results are saved and used in the neck speaker.

Product No.: ABH-130

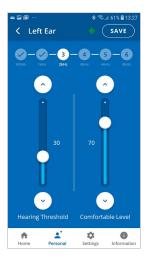

Using BeHear PROXY

Listening to Audio

You can listen to audio from your TV set, mobile phone, or any other paired and connected Bluetooth device using your BeHear PROXY neck speaker.

Use the buttons on the right side of the neck speaker to handle audio play as follows:

- Press | I to play an audio source from the connected Bluetooth device. To pause the source press | I again.
- To increase audio volume press + briefly. A long tone will be heard once you reach the maximum volume level.
- To decrease audio volume press briefly. A long tone will be heard once you reach the minimum volume level.
- Press and hold down + to forward to the next audio track.
- Press and hold down to return to the previous audio track.
- To switch between two connected audio sources (for example, between the HearLink PLUS transmitter and a smartphone) press and simultaneously.

SonicVibrance – Stereo Sound Enhancement Function

When you set your BeHear PROXY neck speaker to **loudspeaker mode** (by placing the mode switch in **)** position), you can enjoy of enriched stereo sound while listening to audio. The embedded SonicVibrance function processes the music to achieve a stereo effect and improve the clarity of the sound.

You can turn SonicVibrance OFF and ON using the **W&H BeHear** app.



Use the neck speaker's multifunction button to manage SonicVibrance:

To activate and deactivate sound processing during audio play press and hold down the
 ▶II button.

Note: Ensure that the **SonicVibrance Control** function of the **Button Functions** is enabled in the **Settings->Device** menu of the **W&H BeHear** app.

ListenThrough™ – Stay Safe & Alert Function

When you set your BeHear PROXY neck speaker to the **earphone mode** (by placing the mode switch in \bigcap position), you can stay aware of important ambient sounds while listening to audio through the earphones . The embedded ListenThroughTM feature passes ambient sounds to your ears even while the earphones are in your ears.

You can manage ListenThrough using the **W&H BeHear** app, as follows:

- Use the slider to set the desired volume of the ambient sound by touching the round button on the scale and dragging it left (softer) or right (louder).
- Set the round button on the slider to the far left to deactivate ListenThrough. Moving the round button to the right reactivates ListenThrough.



Use the neck speaker's multifunction button to manage ListenThrough:

• To mute and unmute ambient sound during audio play press and hold down the button. Voice prompts are played indicating whether ListenThrough mode is ON or OFF.

To define the multifunction button set the **ListenThrough Control** of the **Button Functions** to ON in the **Settings->Device** menu of the **W&H BeHear** app.

Handling Mobile Phone Calls

You can manage mobile phone calls in either neck speaker operation mode: — earphone or loudspeaker mode.

Use the buttons on the right side of the neck speaker to handle mobile phone calls, as follows:

- To **answer** a call press when you hear an incoming call signal.
- To **end** a call press | during an ongoing call.
- To reject a call press and hold down when you hear an incoming call signal.
- To **redial** the last dialed phone number double click .

Note: Ensure that the **Call Control** function of the **Button Functions** is enabled in the **Settings-** >**Device** menu of the **W&H BeHear** app.


- To increase call volume press + briefly. A short tone will be heard indicating change of volume. A long tone will be heard once you reach the maximum volume level.
- To decrease call volume press briefly. A short tone will be heard indicating change of volume. A long tone will be heard once you reach the minimum volume level.

EasyListen™ - Slowing Down Incoming Speech

You can slow down incoming speech for mobile phone calls, making it more intelligible. The **EasyListen** function can be controlled from either the **W&H BeHear** app or the neck speaker.

Enable EasyListen and adjust the speech tempo using the **W&H BeHear** app, as follows:

- Once the phone call has been established, bring the W&H BeHear app to the foreground on your phone.
- Tap the Slow, Slower or Slowest button to activate EasyListen and set the desired speech tempo. A voice prompt will be heard once your choice is accepted.
- Tap the Normal button to deactivate EasyListen. Speech tempo reverts to normal.

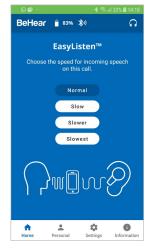

You can define the buttons of the neck speaker to handle EasyListen:

- To activate EasyListen press and hold down the that the Slowest speech tempo is set automatically when EasyListen is activated.
- To deactivate EasyListen press and hold down the button. A voice prompt indicates that the speech tempo has returned to Normal.

Note: Ensure that the **EasyListen Control** function of the **Button Functions** is enabled in the **Settings->Device** menu of the **W&H BeHear** app.

Personal Hearing Amplification

When you set your BeHear PROXY neck speaker to **earphone mode** (by placing the mode switch in position), the neck speaker operates as a personal sound amplifier. Ambient sounds are amplified and transmitted from the microphone directly to your ears. Bluetooth pairing is not mandatory for this type of sound amplification.

RECOMMENDATION:

For best results we recommend turning off key tones (alerts, notification sounds, keyboard taps, etc.) in the smartphone connected to your BeHear PROXY neck speaker. Alternatively, place the phone in "vibrate" or "silent" mode. This will guarantee smooth and pleasant operation of your BeHear PROXY as a personal sound amplifier without undesired sound disruptions produced by background smartphone activity.

The default configuration of the personal sound amplifier supports optimal hearing for a typical home environment. This configuration can be used as is. It will be personalized automatically based on the results of the BeHear PROXY **Hearing Assessment**, which is included the **W&H BeHear** smartphone app. Furthermore, the configuration of the sound amplifier can be customized by the user in real-time to suit specific hearing situations. The customized configuration can either be saved as the new default or discarded.

Managing Personal Hearing Using the Neck Speaker Controls

Use the buttons on the right side of the neck speaker to manage Personal Hearing Amplification:

- To activate Personal Hearing Amplification, press and hold down ▶|| . A voice prompt will be heard as confirmation that Personal Hearing has been activated. To turn off Personal Hearing amplification press and hold down ▶|| again. A voice prompt indicates that Personal Hearing has been deactivated.
- To increase volume of ambient sound press + . A short tone will be heard indicating change of volume. A long tone will be heard once you reach the maximum volume level.
- To decrease volume of ambient sound press _____ . A short tone will be heard indicating change of volume. A long tone will be heard once you reach the minimum volume level.

Note: Ensure that **Hearing Amplification Control** function of the **Button Functions** is enabled in the **Settings->Device** menu of the **W&H BeHear** app.


Managing Personal Hearing Using the W&H BeHear App

- Use the slider to set the desired volume of the ambient sound by touching the round button on the scale and dragging it left (softer) or right (louder).
- Setting the slider to the far left will deactivate Personal Hearing amplification. Moving the slider to the right will reactivate amplification.
- Tap the large round button to enter the advanced Sound Tuning screen.



Advanced Sound Tuning

- Control the level of **Background Noise** by touching the round button on the scale and dragging it left (**Min**) or right (**Max**).
- Set the desired Balance between left and right ear for ambient sounds by touching the round button on the scale and dragging it Left or Right.
- To find the most comfortable listening level for the current environment, activate the **Sound Tuning** option by sliding the round button to the right.
 - Then, touch and hold a square in the **Sound Tuning** grid with your finger until you feel the vibration. Wait for 1-2 seconds until the change is applied (a white frame will appear around the relevant square). Repeat this with other squares to find the position where the hearing is most pleasant to your ears. The Y axis represents changes in amplification (from low at the bottom to high at the top) and the X axis represents different sound frequencies. The dark blue arc indicates the effect of your choice on the amplification level for all the frequencies in the grid.
- You can save your settings by tapping SAVE, or restore the factory defaults by tapping RESET.

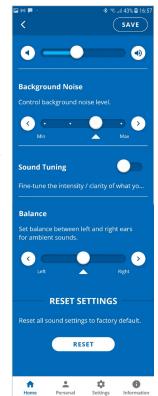



Note: If you deactivate Personal Hearing before saving your changes, your changes will be lost.

Mobile Application

Installing the Application

For Android Phones

- 1. Launch the Google Play application and search for **W&H BeHear**
- 2. Download, install and launch the application.





For iPhones

- 1. Launch the App Store application and search for W&H BeHear.
- 2. Download, install and launch the application.





First Launch

When the app is first launched after download you will be prompted to choose a desired language for the interface, and accept the terms of use.



If no BeHear devices are connected to the phone, the app will continue to the **Connection** screen. Otherwise, you will be prompted to perform the **Hearing Assessment**.






Connection

To manage the pairing and connection of your BeHear PROXY neck speaker with a Bluetooth device, such as a smartphone, tap the **Settings** icon at the bottom of the app screen and choose the **Connection** option.

The app opens the Connection screen automatically when no BeHear PROXY neck speaker is connected to the phone.

You can display the Connection screen at any time by tapping the **Settings** icon at the bottom of the screen and choosing the **Connection** option.

Upon successful connection of a new neck speaker you will be prompted to select the desired language for the voice prompts. Once you have made your choice you will be directed to perform the Hearing Assessment.

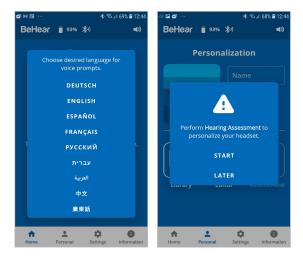


Operation of the Connection screen is different for Android and iOS phones, as described in the following sections.

WARNING:

The **Home** screen icon, as well as parts of other screens, are disabled (grayed out) when the BeHear PROXY neck speaker is not connected to the phone. Connect the neck speaker to a smartphone get full functionality of the application.


Connection with an Android Phone

- 1. Tap the **BeHear PROXY** icon in the Connection screen to initiate connection with your neck speaker.
- 2. Follow the on-screen animated instructions to place your neck speaker into <u>Bluetooth</u> pairing mode:
 - Make sure the BeHear PROXY neck speaker is powered OFF.
 - Then, shift and hold the \circlearrowleft power button on the neck speaker for **up to 5** seconds, until the LED alternates red/blue.
- 3. Wait until the list of Available devices appears in the app. It may take up to 30 seconds.
- 4. The product name **BeHear PROXY** followed by the last several characters of the device's serial number will appear in the list of **Available** devices. You can verify the device by checking that the serial number printed on the underside of the BeHear PROXY neck speaker (on the white label) matches the number in the list.
- 5. Select your BeHear PROXY neck speaker in the list by tapping the device name. When the pairing and connection process is complete the neck speaker will be listed in the **Paired** devices section, followed by a ✓.
- 6. You can manage connected devices in the **Connection** screen as follows:
 - To connect a new BeHear device, tap ADD NEW button.
 - To **connect a previously paired** BeHear PROXY, tap the name of the device in the list.
 - To **disconnect** BeHear PROXY tap the name of the connected device in the list. To **reconnect** BeHear PROXY tap the name again.
 - To unpair BeHear PROXY tap the icon.

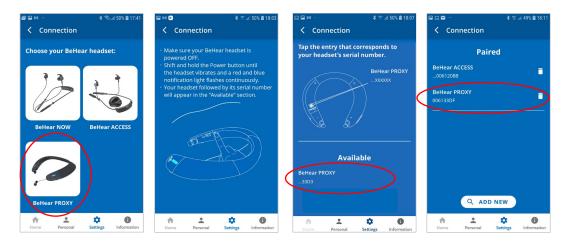


Connection with an iOS Phone

- 1. Tap the **BeHear PROXY** icon in the Connection screen to initiate connection with your neck speaker.
- 2. Tap **NO** if your neck speaker is not paired with the iPhone, otherwise tap **YES**.
- 3. When your selection is **NO**:
 - a. Follow the on-screen animated instructions to place your neck speaker into <u>Bluetooth</u> pairing mode:
 - Make sure the BeHear PROXY neck speaker is powered OFF.
 - Then, shift and hold the power \circlearrowleft button on the neck speaker for **up to 5 seconds**, until the LED alternates red/blue.

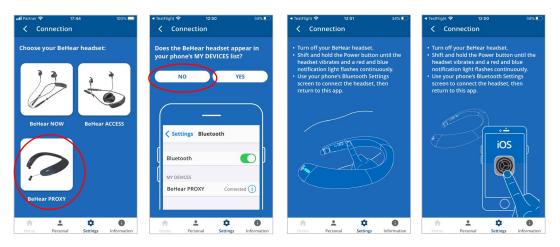


- b. Use the Bluetooth section of the **Settings** application on your phone to add BeHear PROXY to the list of **My Devices**.
- c. Return to the W&H BeHear app and tap **YES** in the connection screen.

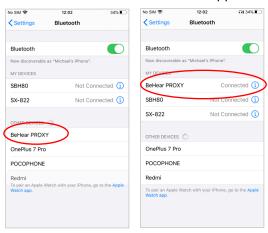

4. When your selection is YES:

- a. The product name BeHear PROXY followed by the last several characters of the device's serial number will appear in the list of Available devices. You can verify the device by checking that the serial number printed on the underside of the BeHear PROXY neck speaker (on the white label) matches the number in the list.
- b. The **Bluetooth Pairing Request** prompt appears.
- c. Select **Pair** and wait for 2-3 seconds to complete the synchronization.
- d. When the pairing and connection process is complete the neck speaker will be listed in the **Paired** devices section, followed by a \checkmark .

Note: If the time between pairing the neck speaker in the Settings application and launching **the** W&H BeHear app is longer than two minutes you will be prompted to double click the Power button of the neck speaker to continue with the connection process.

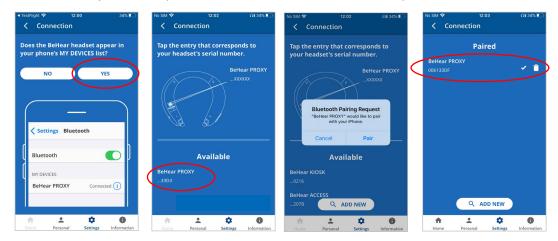

- 5. You can manage connected devices in the **Connection** screen as follows:
 - To connect a new BeHear device tap the ADD NEW button.
 - To connect a previously paired BeHear PROXY tap the name of the device in the list.
 - To **disconnect** BeHear PROXY tap the name of the connected device in the list. To **reconnect** BeHear PROXY tap the name again.
 - To **unpair** BeHear PROXY tap the icon.
 - To **unpair** BeHear PROXY from your phone delete both records of the BeHear PROXY in the Bluetooth section of the **Settings** application.

Home Screens

The application's Home screen changes automatically depending on the actual state of the neck speaker (personal hearing amplification, phone call handling, or audio play).


Tap the **Home** icon in the bottom menu to return to the Home screen from anywhere in the app.

ListenThrough Audio Player State

This state of the Home screen appears during music play while BeHear PROXY is set to the **earphone** mode (the mode switch in position).

In this state the Home screen supports managing the ListenThrough function of the BeHear PROXY working as a stereo audio player. Available control functions are:

 Ambient sounds volume control slider with an option to turn the ListenThrough feature ON and OFF.

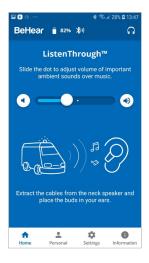

SonicVibrance Audio Player State

This state of the Home screen appears during music play while BeHear PROXY is set to the **loudspeaker** mode (the mode switch in position).

In this state the Home screen supports managing the SonicVibrance function of the BeHear PROXY working as a stereo audio player. Available control functions are:

• ON/OFF switch to turn the SonicVibrance feature ON and OFF.



Phone Call State

In this state the Home screen supports managing the EasyListen function of the BeHear PROXY, working as a neck speaker during the phone call. The state appears in any operation mode of the neck speaker.

Available control functions are:

Speech tempo selection buttons – Normal (EasyListen disabled),
 Slow, Slower, Slowest



Personal Hearing Amplification State

This state of the Home screen appears when no music play or phone call is active while BeHear PROXY is set to **earphone** mode (the mode switch is in position).

In this state the Home screen supports managing BeHear PROXY as a personal sound amplifier.

Available control functions are:

- Hearing volume control slider with an option to turn hearing amplification OFF and ON.
- Tuning button to manage advanced sound tuning options.



Idle State

This state of the Home screen appears when no music play or phone call is active while BeHear PROXY is set to the **loudspeaker** mode (the mode switch is in **4**) position).

There are no control functions available in this state.



Advanced Sound Tuning for Personal Hearing Amplification State

✓ Volume

This control sets the desired volume of the ambient sound by touching the round button and dragging it left (softer) or right (louder).

✓ Background Noise

This control sets the level of Background Noise. Touch the round button and drag it left (**Min**) or right (**Max**).

The recommended setting (factory default) is indicated by a triangle.

✓ Sound Tuning

This control allows you to adjust sound parameters for the Personal Hearing Amplification.

Slide the round dot to the right to activate this control option. Then, touch and hold any square in the grid until you feel a vibration under your finger. The selected square will be framed in white, and relevant sound changes will be applied to the neck speaker.

Moving to the right from the center position increases the high frequency signals, while moving to the left increases the low frequency signals. Moving up and down increases and decreases sound gain, respectively.

The dark blue arc indicates how the modified selection affects the current audiogram.

Return to the recommended position (factory default) by tapping and holding the **Center** square.

✓ Balance

This control sets the desired amplification balance between left and right ears. This control manipulates the balance beyond what was determined by the hearing assessment.

The setting is applied to the volume control – in addition to the results of the built-in hearing assessment – and will be active in personal hearing mode only.

Note: To set the balance for phone calls and music use **Balance** control located in the **Settings->Device** menu.

Slide the round dot to set the desired amplification balance between left and right ears.

Touch the triangle to reset the dot in the middle of the slider to return it to the levels determined by the hearing assessment, if performed.

Note: The hearing amplification is disabled temporarily while moving the slider in the neck speakers with software release below 0408.

✓ SAVE

The **SAVE** button saves the **Sound Tuning** and **Background Noise** settings. Other parameters are saved automatically once adjusted.

✓ RESET SETTINGS

The **RESET** button restores all sound tuning settings to their factory default values.

Product No.: ABH-130





Personalization

Tap the **Personal** icon in the bottom menu to access the Personalization options.

✓ Library

Tap the **Library** button to open the library of audiograms.

✓ Editor

Tap the **Edit** button to open the editor where personal audiogram values can be entered and modified manually.

✓ Assessment

Tap the **Assessment** button to start the Personal Hearing Assessment (to personalize your neck speaker).



Note: The **Assessment** button is available when the neck speaker is in the earphone mode only (the mode switch is in position).

✓ User Name

Unique names are generated and saved as the User Name for each personal audiogram saved in the library. You can edit the name by tapping on it.

✓ Actual Audiogram

The displayed chart depicts the current audiogram values of the connected BeHear PROXY neck speaker. Tap the chart to display the audiogram in full screen mode.

Here you can:


- ✓ View the current values
- ✓ Tap **RESET** to reset the values and restore the default presets
- ✓ Tap **EDIT** to enable manual editing of the values

Personal Hearing Assessment

Choose the **Assessment** option in the menu to access the hearing assessment screens. Initial screens of the Hearing Assessment allow you to:


- Start the assessment
- Resume a previously interrupted assessment, or discard it and start a new assessment.


You can perform the assessment only if you are in a quiet environment. It is not possible to perform the assessment if the ambient noise level (as determined by the app) is too high.

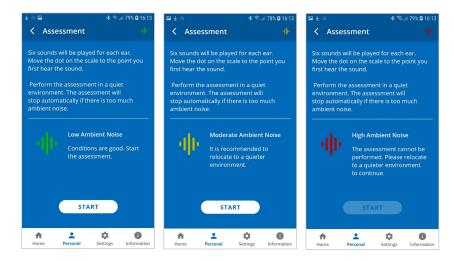

If the **Basic** assessment has been selected in the Device Settings:

- The assessment screens contain a single slider.
- In order to progress with the assessment, you must tap SAVE (at the right top corner of the screen) after setting the slider for each frequency.
- The assessment results shown after completing the assessment display a single chart for the hearing threshold results (for both ears).

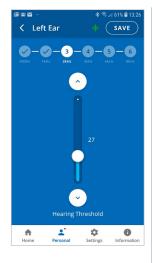

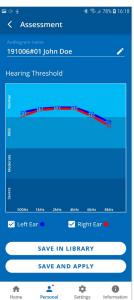

If the **Advanced** assessment has been selected in the Device Settings:

- The assessment screens contain two sliders.
- In order to progress with the assessment, after setting the slider for each frequency you must tap **SAVE** (at the right top corner of the screen). The assessment results shown after completing the assessment display two charts, one for Hearing Threshold and the other for Comfortable Level. To display the relevant results tap the chart name.




To save the assessment results and apply them to the neck speaker:

- Tap **SAVE IN LIBRARY** to store the results in the app library. These results may be retrieved and applied to the neck speaker at a later time. The file name is created automatically.
- Tap SAVE AND APPLY to store the results in the app library and apply the personalized parameters to the connected BeHear PROXY neck speaker.
- Choose **APPLY NOW** in the notification dialog to apply the results to the BeHear PROXY neck speaker. The neck speaker will shut down automatically. You must turn on the neck speaker manually following notification on the screen to complete the upload process.
- If the **POSTPONE UPDATE** option is selected, the neck speaker and the app will continue to operate normally (without applying the results of the assessment until the neck speaker is turned off). The warning icon A, along with a red dot near the **Personal** icon will remind

you that you have a postponed action. Results of the last assessment are applied automatically when the neck speaker is turned back on.



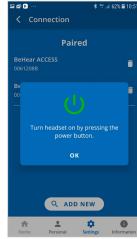




Edit Audiogram


The Editor allows you to create a new audiogram by entering hearing loss values manually (typically from the results of a hearing test performed elsewhere). Modifications of existing audiograms can be done here as well.

- Choose Editor in the Personalization screen to open an empty table for inputting values. The automatically created file name can be changed by tapping on it.
- Input the hearing loss values as they appear in your external audiogram.
- Tap SAVE IN LIBRARY to store the file or SAVE AND APPLY to store the file and apply these personalized parameters to the connected BeHear PROXY neck speaker.
- To modify an existing audiogram, tap **EDIT** while viewing the relevant chart.



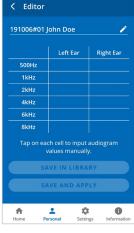



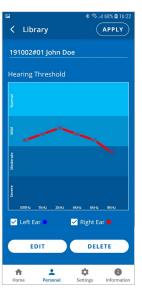

Library of Audiograms

The app library contains factory default audiograms as well as audiograms you have saved.

Choose **Library** in the Personalization screen to open the gallery of audiograms.

Product No.: ABH-130

- Tap a chart in the list to display its values in full screen.
- Tap **APPLY** to upload the audiogram to your connected BeHear PROXY neck speaker.

- Tap EDIT to manually edit the existing values.
- Tap **DELETE** to remove this user audiogram from the library. The default audiograms cannot be deleted.



Settings

Customization of both the neck speaker and the app is done via the Settings screen. Display the Settings screen by choosing the **Settings** icon in the bottom menu.

✓ Connection

Tap the **Connection** button to manage the Bluetooth connection of BeHear PROXY neck speaker with the app.

Refer to the <u>Connection section</u> of this manual for detailed explanations.

✓ Application

Tap the **Application** button to manage app settings.

✓ Device

Tap the **Device** button to configure the connected BeHear PROXY neck speaker.

✓ Update

Tap the **Update** button to manage firmware updates for the connected BeHear PROXY neck speaker.

Product No.: ABH-130



Application Settings

✓ Language

This setting defines the language of the app interface.

√ Hearing Assessment Mode

This setting defines the type of hearing assessment to be performed:

- **Basic** the assessment contains one measurement (hearing threshold) for each ear.
- Advanced the assessment contains two measurements (hearing threshold and most comfortable level) for each ear.

The recommended setting (application default) is **Basic**.

√ Headset Update Check

This setting determines how checks for neck speaker firmware updates will be performed:

- Auto app checks for updates automatically
- Manual user needs to enter Headset Update screen and check for an available update

The recommended setting (application default) is Auto.

✓ Headset Update Download

This setting determines the preferred network connection when downloading updates for the neck speaker firmware:

- **Wi-Fi only** the app performs the neck speaker update automatically only when the phone is connected to the Internet via local Wi-Fi network. A warning notice is displayed when you initiate the update when a Wi-Fi connection is not available.
- Any Network app performs the neck speaker update when the phone is connected to the Internet via any network – Wi-Fi or cellular.

The recommended setting (application default) is Wi-Fi only.

✓ Notifications

This setting controls the behavior of warnings and notifications about unsaved changes to hearing parameters made in the Personal Hearing screen.

Product No.: ABH-130

The recommended setting (application default) is OFF.

✓ Reset Settings

Tap the **RESET** button to restore application defaults.



Device Settings

BEHEAR FEATURES

✓ Personal Hearing Amplification

This setting defines the automatic activation behavior for the Personal Hearing function:

- ON automatically turned on when there is no active phone call or music/audio playback
- OFF automatically turned off when there is no active phone call or music/audio playback
- Last State automatically turned on or off according to the last actual state of the device before handling a call or playing a music/audio

The function can be turned on or off manually in real time either by pressing and holding on the neck speaker or by moving the slider on the Personal Hearing screen in the application.

The recommended setting (device default) is **ON**.

✓ ListenThrough™ Music

This setting defines the automatic activation behavior for the ListenThrough function of ambient sound passthrough while listening to audio, as follows:

- ON automatically turned on when music playback starts
- **OFF** automatically turned off when music playback starts
- Last State automatically turned on or off according to the last choice during the previous music playback
- The function can be turned on or off manually in real time either by pressing and holding the button on the neck speaker or by moving the slider on the ListenThrough screen in the application.

The recommended setting (device default) is **OFF**.

✓ EasyListen™ Comprehension

This setting defines the automatic activation behavior for the EasyListen function of slowing down speech tempo during phone calls, as follows:

- **ON** automatically turned on when phone call starts
- OFF automatically turned off when phone call starts
- Last State automatically turned ON or OFF according to the last choice during the previous phone call

Product No.: ABH-130

The recommended setting (device default) is OFF.



NOTIFICATIONS

√ Voice Prompts Language

Choose the desired language for voice prompts in the BeHear PROXY neck speaker. The list of available languages may vary depending on the hardware and software versions of the device.

✓ Voice Prompts Volume

This control sets the desired volume of the voice prompts by touching the round button and dragging it left (softer) or right (louder).

✓ Notification Tones Volume

This control sets the desired volume of the notification tones by touching the round button and dragging it left (softer) or right (louder).

✓ Vibration

This setting controls vibration notification behavior in the BeHear PROXY neck speaker for the following events:

- Headset power on and power off
- Bluetooth pairing connection/disconnection
- Connecting to the charger
- Incoming call (synchronized with call tone)

The recommended setting (device default) is **ON**.

SOUND

✓ Personalized Music

This setting enables or disables sound personalization according to the hearing assessment results for music listening.

The recommended setting (device default) is **ON.**

✓ Personalized Call

This setting enables or disables sound personalization according to the hearing assessment results for mobile calls.

The recommended setting (device default) is ON.

✓ Balance

This control sets the desired amplification balance between left and right ears.

The setting is applied to the volume control – in addition to the results of the built-in hearing assessment – and will be active for mobile calls and music only.



Note: To set the balance for personal hearing mode use **Balance** control located in the **Advanced Sound Tuning** menu.

Slide the round dot to set the desired amplification balance between left and right ears. Touch the triangle to reset the dot in the middle of the slider to return it to the symmetrical balance.

Note: This setting is available for the neck speakers with software release number 0408 and higher.

Product No.: ABH-130



ADVANCED

✓ Button Functions

This setting customizes operation of the buttons in the BeHear PROXY neck speaker for the following functions of the manual control:

- Call control
- EasyListen control
- ListenThrough control
- SonicVibrance control
- Personal Hearing Amplification control

Set any function to ON state enables manual control of the function using the buttons as described in this manual.

The recommended settings (device defaults) are:

•	Call control	ON
•	EasyListen control	ON
•	ListenThrough control	OFF
•	SonicVibrance control	OFF
•	Personal Hearing Amplification control	ON

Bluetooth Audio CODEC

This setting selects the desired Bluetooth audio CODEC - APTX LL (Low Latency), APTX or SBC.

- **APTX LL** provides the best performance with minimal latency.
- APTX is similar to APTX LL but the latency is slightly longer.
- **SBC** is the standard Bluetooth CODEC supported by all devices, latency is long and not fixed.

Note: Turn your neck speaker off and then turn it back on to apply the modification of the Bluetooth audio CODEC.

The recommended setting (device default) is APTX LL.

RECOMMENDATION:

Choose APTX or SBC audio if you experience sound distortion while multiple Bluetooth devices are connected.

Product No.: ABH-130

RESET

✓ Reset Settings

Tap the **RESET** button to restore device defaults.






Update

The app supports firmware updates of the connected BeHear PROXY neck speaker, allowing users to enjoy its most advanced features.


You can verify the availability of an update manually by checking the **Update** button in the **Settings** screen.

We recommend choosing the **Auto** option for the **Headset Update Check** in the **Application** section of the **Settings** to get automatic notification whenever a new update is available. In this case, this icon will appear in the notification area.

You can learn about the new firmware version prior starting the update.

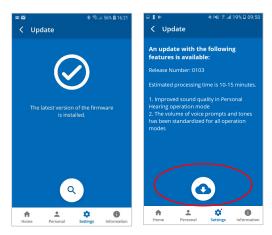

To initiate the process of downloading the firmware and updating the neck speaker tap the **Download** icon. The process will last for several minutes during which time the neck speaker and application cannot be used. Normal operation resumes automatically when the update is complete.

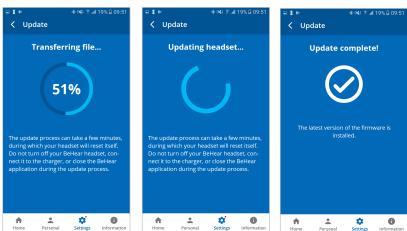

To postpone an update, choose **Later** or exit the **Headset Update** screen. The warning icon will appear at the top of each application screen as reminder to update the firmware. You can initiate an update any time by tapping the **Update** button in the **Settings** screen.

WARNING:

Do not turn off your BeHear PROXY neck speaker, connect it to the charger, or close the W&H BeHear application during the update process.

Information

Information and tutorials about the neck speaker and the application are accessible via the **Information** screen. Display the Information screen by tapping the **Information** icon in the bottom menu.

✓ Introduction

Tap the **Introduction** button to view introduction screens providing brief descriptions of the major BeHear PROXY functions.

✓ Tutorials

Tap the **Tutorials** button to watch YouTube video tutorials online.

✓ Version

Tap the **Version** button to check information about the application and the connected BeHear PROXY neck speaker.

✓ Terms of Use

Tap the **Terms of Use** button to review the legal terms of use for the application and connected BeHear PROXY neck speaker.

✓ FAO


Tap the **FAQ** button to find answers to frequent customers' questions in the Wear&Hear website.

✓ Support

Tap the **Support** button to find contact information for technical support.

Introduction

The introduction screens provide brief descriptions of the main BeHear PROXY functions, such as Personal Hearing Amplification, EasyListen™ Phone Speech Comprehension, and ListenThrough™ Music Appreciation.

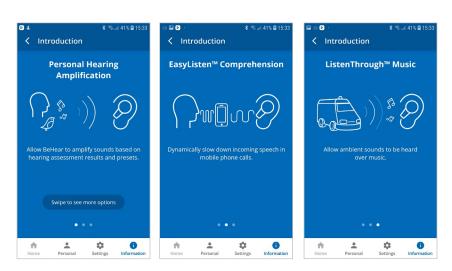

Navigate between the introduction screens by swiping the phone's screen.




Tutorials

Select **PROXY** in the list of products to watch YouTube video tutorials for BeHear PROXY neck speaker online.

An Internet connection is required to get access to the videos.



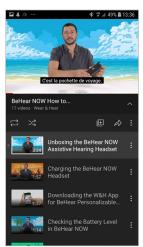

Version

Display the information about the W&H BeHear application and connected BeHear PROXY neck speaker by choosing the **Version** option in the Information screen.





Terms of Use

Review legal terms of use for the W&H BeHear application and connected BeHear PROXY neck speaker by choosing the **Terms of Use** option in the Information screen.



FAQ

Find answers to frequent customers' questions in the Wear&Hear Web site by choosing the **FAQ** option in the Information screen.



Support

- ✓ Find contact information for technical support by choosing the Support option in the Information screen.
- ✓ Send device and app information to support service by tapping SEND button.



Troubleshooting

General Issues

Problem	Suggestion
No sound	Verify battery level (see <u>Checking the Battery Status</u>)
	Adjust volume level (see Managing Personal Hearing Using the Headset
	Controls)
	Check ear caps for debris
Ear buds are	Experiment with different silicon ear buds (see Fitting the Ear Buds to Your
uncomfortable	Ears)
Ambient noise too loud	Reduce the volume of ambient sound (see <u>ListenThrough™ – Stay Safe &</u>
during music play	Alert Function)
Phone calls and/or	Verify Bluetooth pairing (see <u>Pairing with Bluetooth Devices</u>)
music not routed to	
neck speaker	
Bluetooth connection	Verify that you are not more than 10 meters (30 feet) from the Bluetooth
keeps cutting out	device.
	If you are using BeHear PROXY outdoors, we recommend that you carry the smartphone connected over Bluetooth with the BeHear PROXY on the right side of your body.
Personal hearing is	We recommend turning off key tones (alerts, notification sounds,
interrupted by	keyboard taps, etc.) in the smartphone connected to your BeHear PROXY
smartphone notifications	neck speaker It will guarantee smooth and convenient operation of your BeHear PROXY as a personal sound amplifier without undesired sound
Hothications	disruptions produced by smartphone activity.

Reset the Neck Speaker

If your BeHear PROXY behaves unexpectedly you can restart it, remove the Bluetooth connection data and/or restore the original factory configuration.

- **To restart BeHear** simply connect it to the charger. All configuration and connection data will be preserved.
- To restart BeHear and remove the Bluetooth connection data only:
 - Power on the BeHear PROXY neck speaker.
 - Press and hold these three buttons approximately 8 seconds until a purple LED blinks twice.
 - The neck speaker will restart and then enter pairing mode automatically.
- To restart the BeHear PROXY neck speaker and restore the original factory configuration:
 - Power on the BeHear PROXY neck speaker.
 - Press and hold these three buttons approximately 8 seconds until a yellow LED blinks once.
 - The neck speaker will turn off automatically.
 - Power on the neck speaker.

Updating the Neck Speaker

Make sure your BeHear PROXY is using the latest version of the firmware. You can verify availability of an update by manually checking the **Headset Update** option in the menu.

<u>Update the neck speaker</u> if a new version is available, using the W&H BeHear smartphone app.

Legal Information

BeHear[®] is a registered trademark of Alango Technologies, Ltd.

HearLink® is a registered trademark of Alango Technologies, Ltd.

Bluetooth® is the trademark of the Bluetooth (SIG) Inc. and is used under license.

All product and company names mentioned herein are the trademarks or registered trademarks of their respective owners. Any rights not expressly granted herein are reserved.

All other trademarks are property of their respective owners.

FCC Compliance Statement

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment. This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) this device may not cause harmful interference, and
- (2) this device must accept any interference received, including interference that may cause undesired operation.

FCC ID: 2AM3SABH-130

EU Declaration of Conformity

We, Alango Technologies Ltd. of 2 Etgar St. Tirat Carmel, Israel declare under our sole responsibility that our product BeHear PROXY type **ABH-130** and in combination with our accessories, to which this declaration relates is in conformity with the provisions of the following directives:

• Radio Equipment Directive (RED) 2014/53/EU by appropriate standards

ETSI EN 300 328 V2.2.2 (2019-07), ETSI EN 301 489-1 V2.2.3, EN301 489-17 V3.2.4, EN 50332-2:2013, EN 62368-1:2014 + A11:2017, EN 62479:2010, EN50633:2017

 Restriction of Hazardous Substances (RoHS) Directives 2011/65/EU & (EU)2015/863 by appropriate standards

IEC 62321-3-1:2013 / IEC 62321-5:2013, IEC 62321-4:2013 + A1:2017, IEC 62321-7-2:2017 / IEC 62321-6:2015, IEC 62321-7-2:2015 / IEC 62321-8:2017

Reuse / Recycling / Recovery Targets under the Waste Electrical and Electronic Equipment (WEEE)
 Directive 2012/19/EU

Tirat Carmel, December 2020

Alexander Goldin,

CEO

European Union – Disposal Information

This symbol means that according to local laws and regulations this product and/or its battery shall be disposed of separately from household waste. When this product reaches its end of life, take it to a collection point designated by local authorities. The separate collection and recycling of this product and/or its battery at the time of

disposal will help conserve natural resources and ensure it is recycled in a manner that protects human health and the environment.