

Cleaning the Filter of the MARK V Hearing Aid Cleaner

TO CLEAN THE FILTER:

The filter is located on the face of the Mark V Hearing Aid Cleaner. It is between the gauge and the black knob. Remove the clear cap from the filter. It unscrews counter clock wise. Once off, you can see if there is any wax buildup in the white filter. The filter is made of ceramic and is washable. Wash it before leaving for the day and let it dry overnight. Be sure it is completely dry before reinserting.

Once the white filter is out, take a Q-tip and clean the black portion of the filter to remove any wax. When everything is clean and dry, reinsert the white filter and cap. The cap has to be tight to maintain suction. Use a rubber glove for more traction which helps to tighten the cap.

To test the unit for suction, turn the unit on and set the black knob to the wand position. Put your finger over the clear tip of the black tubing. The vacuum gauge should climb to 26-27 inches of mercury. If it does not, tighten the cap more.

