



DIRECTIONS FOR USING HEARING AID DRYING KIT:

1. Before retiring for the night, clean your hearing aids (and ear molds if you have them) with AudioWipes™ cleaning wipes
2. Open battery doors
3. Place the hearing aids on the screen side of the desiccant
4. Replace the light and close tightly

INSTRUCTIONS FOR REACTIVATING DESICCANT DISC:

When all the granules turn white, it is time to reactivate. Remove desiccant disc from the jar and reactivate desiccant in the microwave or the oven

TO REACTIVATE IN MICROWAVE

Place desiccant, screen side up, in microwave and heat for 30 seconds on high. Allow to cool.

IMPORANTANT

- **KEEP OUT OF REACH OF CHILDREN**
- **DO NOT EAT**
- **DO NOT PUT HEARING AIDS IN MICROWAVE**