DryMax[®] UV by Dry & Store[®]

Quick Start Guide










For use with DryMax UV

dryandstore.com/manuals

dryandstore.com/support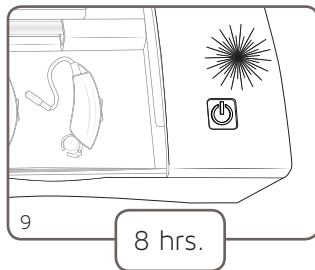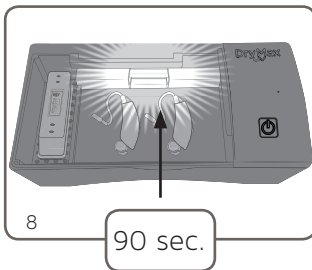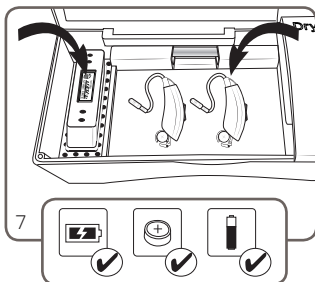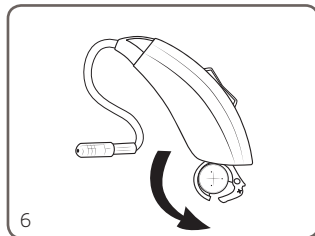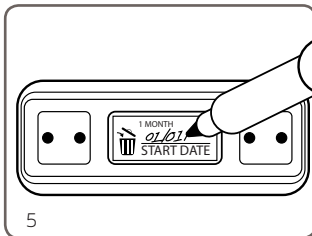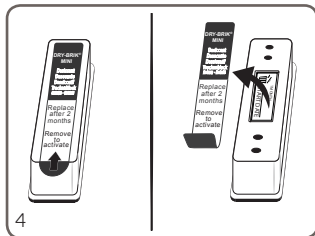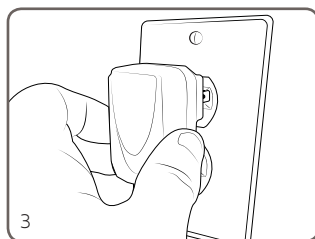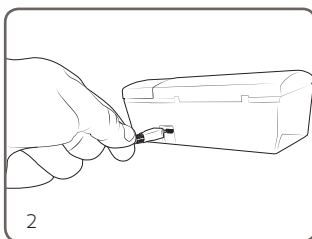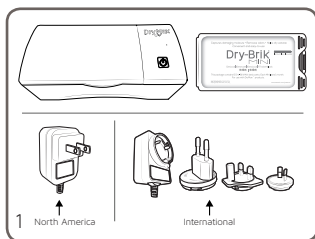

www.dryandstore.com | 1-888-327-1299 | +423-928-9060 | info@dryandstore.com

P.O. BOX 1516 Johnson City, TN 37605

Important Safety Information

IMPORTANT SAFETY INFORMATION

This product is for household use. When using electrical products, especially when children are present, basic safety precautions should always be followed, including:

DANGER—To reduce the risk of electrocution:

- Do not use while bathing.
- Do not place or store product where it can fall or be pulled into a tub or sink.
- Do not place in or drop into water or other liquid.
- Do not reach for this device if it has fallen into water. Unplug immediately.

WARNING—To reduce the risk of burns, electrocution, fire, or injury to persons:

- Closely supervise use by, on or near children or invalids.
- Use this product only for its intended use as described in this guide.
- Never operate this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return it to the place of purchase or service center for examination and repair.
- Never place the product on a soft surface such as a bed. Keep any air openings free of lint, hair, and the like.
- Never use outdoors or where aerosol (spray) products are being used or where oxygen is being administered.
- The AC adapter included with your unit is made specifically for this product. Do not use it with another appliance, and do not use an adapter from any other electrical device in its place.

NOTE: This product is not designed to diagnose, prevent, monitor, treat, or alleviate disease.

OPERATING CONDITIONS At an ambient temperature of 70°F (21°C) the operating temperature of this appliance is 98°F (37°F) to 104°F (40°C).

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DryMax®
by Dry & Store®

Conforms to IEC 60950-1 Ed. 2.0, CISPR 14-1, Ed. 5.1: 2009, CISPR 14-2, Ed. 1.2: 2008
MODEL DMX 150 | 5V DC 0.65 Amper | Micro USB Connector
Made in USA - Johnson City, TN www.aartech.com
COPYRIGHT EART TECHNOLOGY CORPORATION 2017. ALL RIGHTS RESERVED.
Worldwide patents available at www.aartech.com/patents