

## **???** Dry&Clean Mode (2 h)

The *Dry&Clean* mode is an automatic program and combines fast drying by an efficient air blower with UV-C light at the beginning and end of the drying time for perfect hygiene.

### Clean Mode (2 min)

The *Clean* Mode is a short-term hygiene program and starts the optimized UV-C light, which effectively eliminates up to 99.9 % of germs, bacteria and fungi in the household.

## Multiple utility

With the new dry-cap uv 2°, a large number of other products can be quickly dried and hygienically cleansed in addition to rechargeable hearing aids. Here are a few examples from the fields of electronics, dental care, babies and children, cosmetics, valuables and much more...



# dry-cap uv 2°

Electronic drying cap for all rechargeable hearing aids.



Perfect fast drying by air blower

Hygienic cleansing by UV-C light

Simultaneous drying and charging by dual USB power supply

## Dries and maintains your hearing aids while charging


Moisture, body secretions and cerumen can severely impair the function of rechargeable hearing aids and lead to failure.

The new dry-cap uv 2<sup>®</sup> offers optimum care. Fast drying by an efficient air blower significantly extends the service life and reduces repairs.

Ultraviolet light (253.7 nm UV-C) eliminates 99.9% of bacteria, fungi and viruses. Additional blue light is health-improving.

This clinical cleanliness prevents ear infections and protects both your hearing aids and your health. A 3G sensor shuts off the device to protect the eyes when moving.

The Dual-USB power supply allows you to dry, hygienically clean and charge rechargeable hearing aids or other devices (e. g. mobile phones) at the same time from a single power outlet.

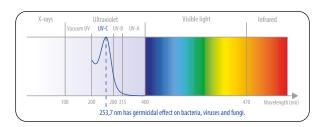



<sup>\*</sup> Suitable for all hearing aid and earphone chargers









blue light

Easy handling by simple touch on



Drying, cleansing and charging by Dual-USB standard

DUAL



**Gravity sensor** 

for safety

shut-off

-2h Automatic turn-off

after 2 hours





microprocessor





2 year warranty

Energy-efficient, gentle and absolutely safe

## The new dry-cap uv 2° is a good choice

The patented dry-cap uv 2<sup>®</sup> as a cap enables drying, hygienic cleansing and charging of rechargeable hearing aids at the same time at only one socket.

Two modes are available to cover a wide range of different applications in the fields of electronics. dental care, babies and children, cosmetics, valuables, etc.

The *Dry&Clean* mode combines fast drying by an efficient air blower and UV-C light for perfect hygiene at the beginning and end of the drying process.

The Clean Mode starts the UV-C light in order to achieve a fast hygiene even with multiple applications.

Whether at home or on the move, the device works on a 5V-USB-basis and can be used in hotels, cars, PCs, TVs or audio devices. A standard USB 2.0 port is sufficient for this.

The new dry-cap uv 2<sup>®</sup> is energy-saving, cost-effective in operation and offers safety in application. Effective and absolutely gentle care is the best guarantee for a long hearing system life.

Your specialist