

# PerfectClean®

The only hearing aid cleaning and dehumidification Medical Device with UV-C

## New version

**Faster, more effective**

Just one year since PerfectClean® being on the market, with thousands of units in use around the world, we introduce the new version of PerfectClean®.

PerfectClean® is now even more reliable and easy to use.




## Key improvements




Drying cycle well within rechargeable battery life



Redesigned housing for better drying

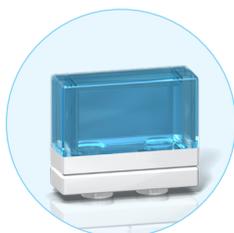

New effective formula to dissolve earwax, and more hypoallergenic

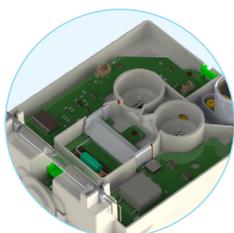


Waterproof electronic card. Design prevents mishandling.

# Features



**Totally removes earwax and impurities**  
thanks to the patented Power Vortex system

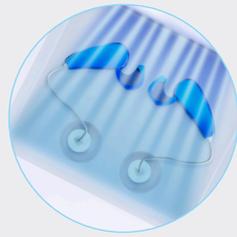

**UV-C lamp**

- Shortwave UV-C (253.7nm)
- No damage to hearing aid surfaces



**Patented, pulsed air drying system**


- Regulated temperature 113°F +/-5°F
- Controlled ventilation

Uniform, optimized air flow

- Removes moisture



**Cleaning solution that dissolves earwax.**  
Hypoallergenic

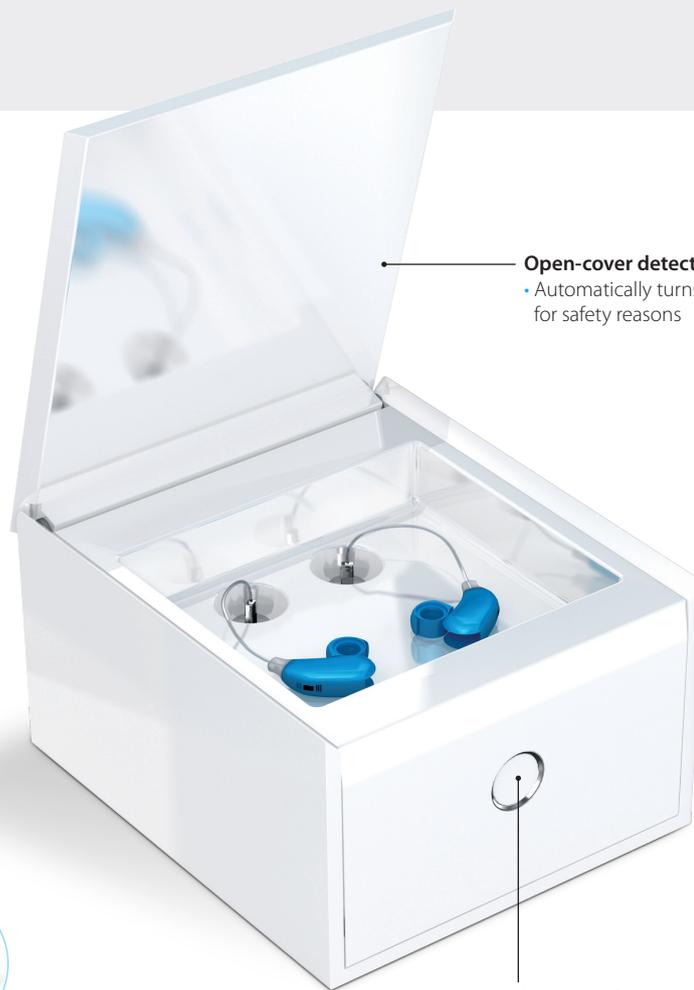


**Open-cover detector**

- Automatically turns off the UV-C lamp for safety reasons

Before cleaning

After cleaning

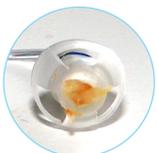

**Indicator light**


• Blue	- flashing slow	> On
• Blue	- steady	> Standby
• Blue	- flashing fast	> Top-up solution
• Red	- flashing fast	> Fault

# Instructions

## Filling


1

Fully remove the refill tab.

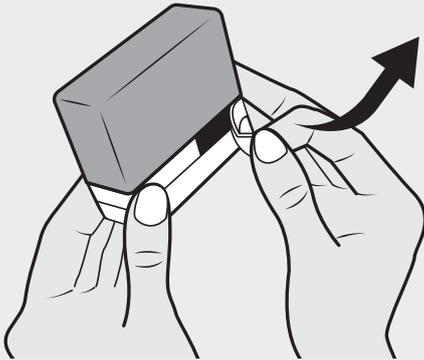


2

Position the refill, top down, in the PerfectClean® refill housing.

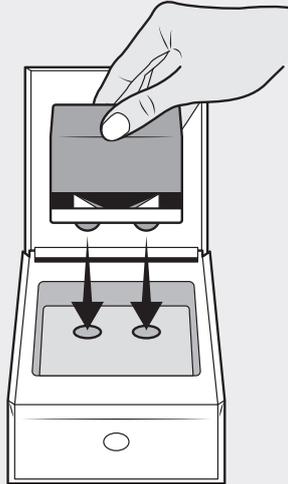

3

Push the refill gradually and horizontally up to the stops to pierce the caps. Wait for all the liquid to run out.

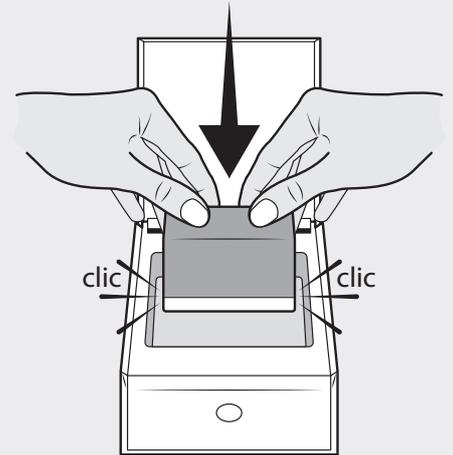

**One refill allows you to complete between 40 and 50 cycles\*, between 20 and 25 cycles\* when the tank is empty (first use or after emptying for transportation)**




\*These quantities are given for your information only. They can vary depending on the hearing aids.

## To start

1

Open the battery compartment (the batteries can remain in their housing), place the hearing aids into the box, placing the tips in the two holes provided. Close the cover.

The PerfectClean® is suitable for all types of hearing aids.

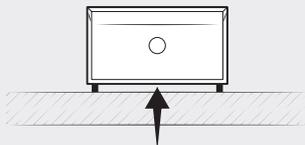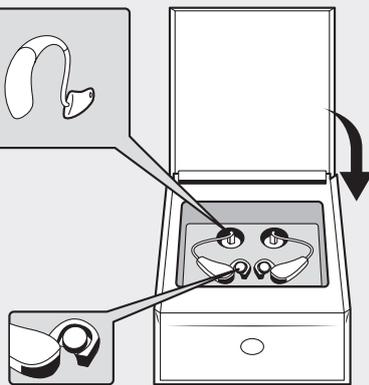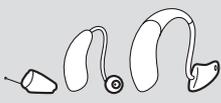

Place PerfectClean® on a smooth, flat surface.  
Do not obstruct the air intake underneath PerfectClean®.

2

Press the touch-button once to launch the cycle (the white flashing light indicates that the request has been processed). The blue indicator flashes slowly, indicating that the box is running. The cycle will only run if the cover is closed. The PerfectClean® stops automatically at the end of the cleaning cycle (1h). The indicator turns off.




30 seconds before starting, the blue LED goes on standby. Open and close the lid again to start the cycle by pressing the touch button.

The new PerfectClean® is available now, and comes with instructions in a more protective, sturdy shipping pack.



Personalisable sleeve

Details inside box

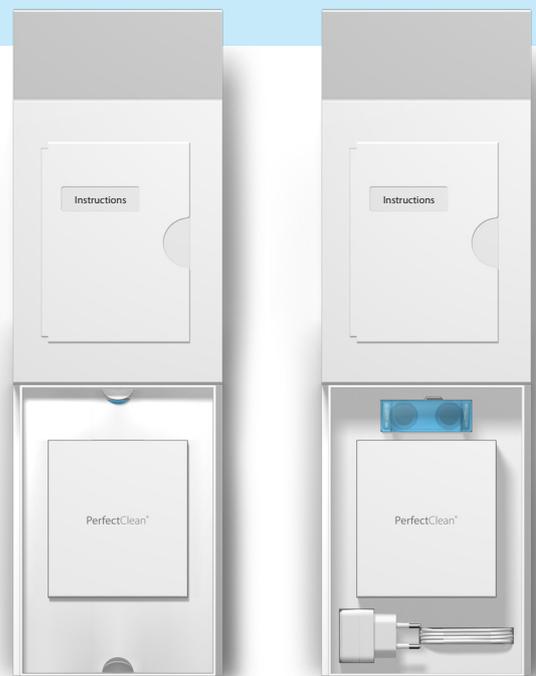

PerfectClean®