

Quick Start Guide

For complete instructions, refer to User Instruction Manual.

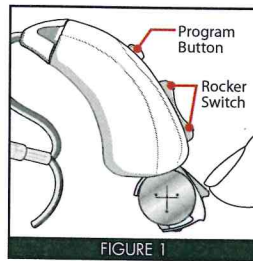


FIGURE 1

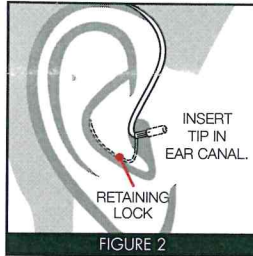


FIGURE 2

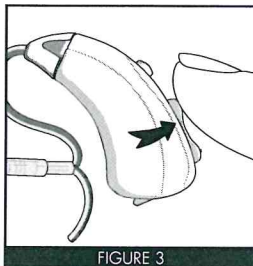


FIGURE 3

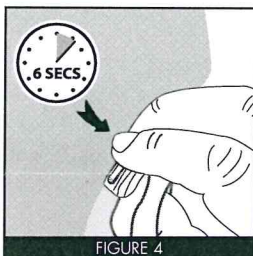


FIGURE 4

1. Prepare for use

Attach either a left or right tube. Insert battery and close the battery door. Your PLAID is now on. (FIGURE 1)

2. Place PLAID over your ear.

Position the tip in the ear canal and the retaining lock in the bowl of your ear. (FIGURE 2)


3. Try out different amplification settings

There are three different amplification settings for you to try.

Start out in a relatively quiet environment. When you first turn PLAID on, it will be at the lowest level of amplification. How does it sound?

Now press and release the upper rocker switch to move to the next amplification setting. (FIGURE 3) You will hear two (2) beeps, indicating you are now at the next setting, and so on, up to three levels of amplification.

Use the rocker switch to go up or down to try out the different settings.

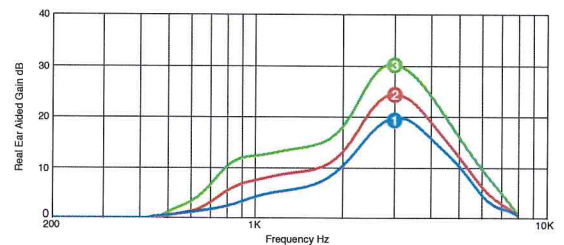

4. Lock in your preferred setting

Once you decide which amplification setting you like best, **press and HOLD the rocker switch for six (6) seconds, until you hear a long beep.** (FIGURE 4)

Congratulations! From this point forward PLAID remembers the Amplification Profile you just set.

Now you can switch between two different listening modes—Quiet and Noise.

To switch between the two listening modes, **press and release the program button.** Once you release it you will hear a confirmation: a single beep for Quiet, and “Shhh” sound for Noise. *In either program, you can still adjust the volume up or down by tapping the upper or lower rocker switch.*

NOTE: If you want more bass sound, remove the sleeve tip and attach one of the dome tips instead. Refer to page 8 in the User Instruction Manual.

Important: To extend battery life, turn off your PLAID by opening the battery door whenever the device is not in use.

PLAID.

www.plaidhearing.com | 1-888-327-1299 | info@eartech.com