

GN



Quick guide
TV-Streamer+

TV-Streamer+


Contents:



TV-Streamer+



Digital (optical) audio cable (remove cable plug covers before use)

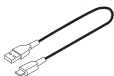

USB-A to USB-C cable
(used to supply power from the TV to the TV-Streamer+)




Analog audio cable



Hook and loop fastening tape strip

Descriptions:

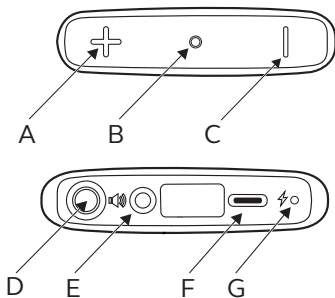

A. Increase volume/delay adjustment

B. LED indicator: audio streaming/pairing indicator

C. Decrease volume/delay adjustment

D. Pairing/delay mode button adjustment

E. Audio input: analog or digital (optical)

F. Power connection: USB-C

G. LED indicator: power





How to set up the TV-Streamer+

Before you start:

Check your hearing aids are paired with the TV-Streamer+.

If not, go to the section “How to pair hearing aids to TV-Streamer+”.

1. Two cables in the box: A digital optical mini-TOSLINK connector (image 1) and an analog stereo 3.5 mm mini-jack to L/R RCA cable (image 2), each with 3.5 mm plug (image 3) on the other end.
2. The TV-Streamer+ has only one 3.5 mm input (image 4) which accepts either analog or digital input.
3. The TV-Streamer+ recognizes which cable is inserted.

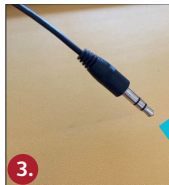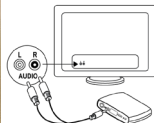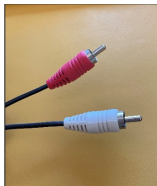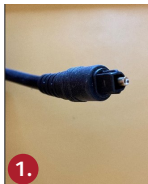

How to pair hearing aids to TV-Streamer+:


1. Place hearing aid(s) within two inches (five cm) of the TV-Streamer+.



2. Press pairing button on the back of the TV-Streamer+.



3. The green LED on the front will flash in short intervals while searching for a hearing aid.

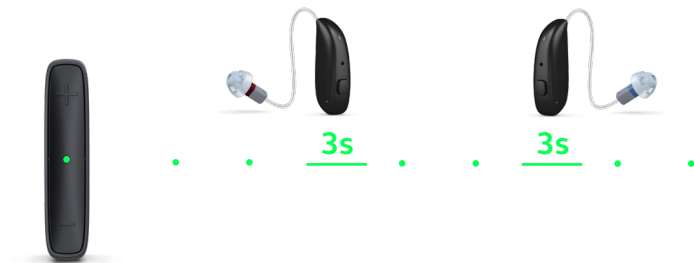

4. Flashing will continue for up to 12 seconds for the first hearing aid, followed by a solid green indicator for three seconds, and the typical hearing aid trill, when paired.

6. Flashing will continue for another 15 seconds as the TV-Streamer+ searches for more devices. If there are no more devices detected, searching will cease.

5. The second hearing aid of the set will then commence pairing automatically.

For more information

Refer to the user guide included in your TV-Streamer+ box. For additional support, please contact the technical support number included in your hearing aid box.

Your hearing care professional can be reached at: _____

Manufacturer according to
EU Medical Device Regulation
2017/745:

GN Hearing A/S
Lautrupbjerg 7
DK-2750 Ballerup
Denmark

Manufacturer according to FDA:

GN Hearing
8001 E Bloomington Freeway
Bloomington, MN 55420
USA



402634011

402634011 revA 2023.09