SuperEar®

Model SE5000

Personal Sound Amplifier

Owner's Manual El manual del propietario Le manuel du propriétaire Bedienungsanleitung Manuale del proprietario

Instructions for Use

1. To remove the battery compartment cover, gently depress the cover with your thumb and slide it in the direction of the arrow.
2. Insert the AAA 1.5v battery. Match the positive (+) and negative (-) ends to those indicated on the battery cavity. Alkaline batteries will deliver 2-3 times the operational life of lesser batteries.
3. Slide the battery compartment cover back into locked position.
4. Insert the earphone connection into the jack.
5. Be sure the amplifier switch is in the OFF position before putting on the earphones. Adjust headband to fit comfortably.
6. Slowly turn the volume control wheel to the desired level of amplification.
7. Attach the pocket clip to the bottom of the amplifier using the screw provided. Do not over-tighten. Can be used on pocket, belt, or other gear.

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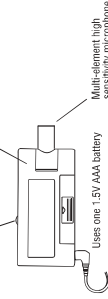
On/OFF switch and volume control wheel

High Impair ABS case

Uses one 1.5V AAA battery

3 YEAR WARRANTY

Sonic Technology Products, Inc. warrants to the original purchaser that your SuperEar® will be free of defects in material or workmanship for a period of 3 years from the date of purchase. Warranty is void if defects are caused by negligence, abuse, misuse or damage caused by water or heat.

This warranty gives you specific legal rights and you may have other rights which may vary from state to state.

Please call us first to determine if it will be necessary to send your SuperEar® in for service under the terms of this warranty. If yes, it must be returned in a well-packaged carton and shipped postage prepaid.

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