

SuperEar® SE5000

SonicTechnology®
Quality and Innovation since 1981

The SuperEar SE5000 has surface mount microelectronics for crystal clear amplification of ambient sounds by up to 50 decibels, for a total decibel gain of 107 dB.



SuperEar® Model SE5000 utilizes (1) AAA Alkaline Battery (Included).

The SuperEar® performs at optimum with **Energizer** or **Duracell** batteries. If static occurs during use, this is typically symptom of battery performance. Changing the battery solves most static issues.

The battery door is located in the front of the unit. Place the bottom of the battery on the spring located in the battery compartment. The left hand side battery has the spring on the bottom/the right hand side has the spring on the top. Please see diagram below.

The Volume Control Wheel is located on the side of the unit. Please always confirm the unit is in the OFF position before placing the headphones on the head. Once the headphone is comfortably positioned, slowly advance the Volume Control to the desired amplification.

Please refer to our cleaning instructions attached for cleaning information and for Disposable Sanitary earpad covers.

Place the Bottom of the Battery on **spring**



Swivel Microphone

Confirm the **OFF** position on the Volume Control Wheel before attaching the headphones to the amplifier.



Disposable Sanitary Earpad Cover



Headphone Jack



Our SuperEar® team is available to help. Contact us:
superear@sonictechnology.com
or 800-247-5548 8am-5pm PST Mon- Thurs.